

Axis Modular Mount System Instructions



CERTIFIED
ADAPTIVE INC.




1. Install Mounting Clamps



- 1.) Install Mounting Clamps on Wheelchair Tubes
- Clamps will fit 3/4", 7/8", or 1" tubes.
* Use provided metal clips for 3/4" and 7/8" tubes.
- 2.) Once desired clamp location is achieved tighten clamps as shown.
- 3.) Mounting Clamps can be moved up and down on tube to adjust back height.


* Quick release insert is already installed on opposite clamp for pin lock installation.

2. Install Mount to Back

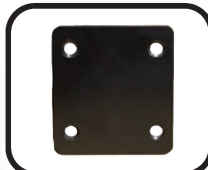

- 1.) Mount Axis Modular System to back cushion through 4-hole mounting block, track mount or rotation mount.
- 2.) If using track, rotation, or track and rotation these components have the same 4-hole configuration for mounting to back cushion and mounting block.
- 3.) All mounting units are completely interchangeable as shown.
- 4.) Axis Modular Mount System will come assembled for easy mounting to back cushion no matter which Axis Mount kit is ordered.

3. Install Quick Release Pin Lock

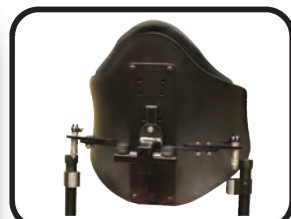

- 1.) After system is mounted to back cushion, simply "drop" the quick release pins into insert on mounting clamp.
- 2.) Once in place close levers on right and left pin locks which will lock back cushion into place.

* System can be completely removed by unlatching quick release and pulling system from inserts.

4. Rotation/Coronal-Sagittal-Transverse Adjustment



- 1.) Loosen screws as shown to adjust width medially/laterally.
- 2.) Channels on wings allow for "slide" adjustment in the coronal, sagittal, transverse planes on both sides.
- 3.) Wing/slide design also allows rotation for asymmetries in coronal, sagittal, and transverse planes.
- 4.) Once desired position is achieved - tighten screws in place.

* All other adjustments can still be achieved no matter the seat depth position set.


5. Seat Depth Adjustment



- 1.) Loosen serrated lock on both sides to adjust seat depth.
- 2.) Once desired depth is achieved tighten screw in place.

* All other adjustments can still be achieved no matter the seat depth position set.

6. Seat to Back Angle Adjustment

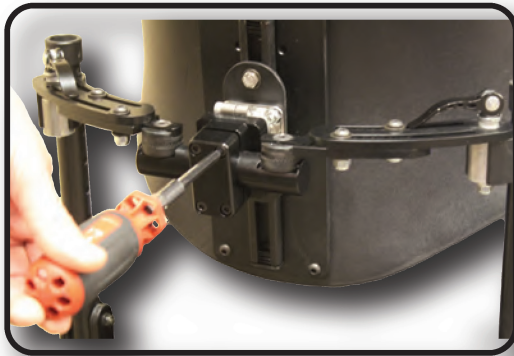

- 1.) Loosen center mount screws and rotate mount on round bar to adjust seat to back angle.
- 2.) Once seat to back angle position is achieved tighten screws in place to back round bar.

* All other adjustment can still be achieved no matter the seat to back angle position set.

7. Track/Height Adjustment



- 1.) Loosen track bolts on top and bottom and slide on track until desired height is achieved.
- 2.) Tighten top and bottom bolts in place.
- 3.) Attach track to back cushion unit through metal strap holes as shown.

* All other adjustment can still be achieved no matter the height set on the track.

8. Rotation Adjustment



- 1.) Loosen set screw on rotation unit as shown which will allow infinite rotation in either direction.
- 2.) Once desired rotation position is achieved tighten set screw in place to maintain position.

* All other adjustment can still be achieved no matter the rotation position set.



CERTIFIED

ADAPTIVE INC.

Axis Modular Mount System Instructions



Axis Modular Mount System is manufactured by:

Certified Adaptive
1709 Heath Parkway
Fort Collins, CO 80524
Phone: 1 (800) 466-7015 or (970) 482-7116
Fax: (970) 498-9529
Email: info@certifiedortho.com
www.certifiedadaptive.com

